F4H5[®] WashOut

Deutsch	1
English	2
Français	3
Italiano	4
Español	5
Português	6
Ελληνικά	7
Nederlands	8
Dansk	9
Norsk	10
Svenska	11
Suomi	12
Eesti keel	13
Latviešu	14
Lietuviškai	15
Íslenska	16
Polski	17
Česky	18
Slovenčina	19
Magyar	20
Slovensko	21
Română	22
Български	23
Русский	24

F4H5® WashOut

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F4H5® WashOut

Ultrapure semifluorinated alkane

English | 2

F4H5® WashOut is a colourless, homogeneous liquid that consists of 100 % perfluorobutylpentane and no biologically active components. F4H5® WashOut is chemically and physiologically inert and non-toxic. F4H5® WashOut produced by Fluoron GmbH is characterised by consistently high quality.

Indications

F4H5® WashOut is used as a medical aid, especially in cases of:

- For the removal of silicone oil residues from the vitreous chamber
- For cleaning intraocular lenses following silicone oil tamponade


Composition

100 % perfluorobutylpentane ($C_4F_9C_5F_{11}$)

Physical properties

Device	Density [kg/m ³ , 35 °C]	Refractive index [35 °C]
F4H5®	1270	1.32

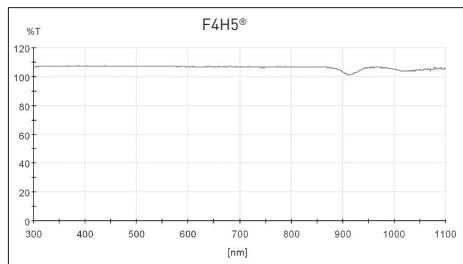Spectral transmission

Operating instructions

F4H5® WashOut is intended only for ocular application; it is applied at a dose determined by an ophthalmic surgeon for a defined indwelling time.

Preparation

F4H5® WashOut in vials is drawn up undiluted into a sterile syringe with a suitable cannula attached.

F4H5® WashOut is injected directly into the eye following removal of a silicone oil tamponade. Standard cannulas are able to be used for the injection. The required quantity must be determined by the treating doctor. F4H5® WashOut must have contact with the silicone oil drops to be dissolved in order to ensure removal. F4H5® WashOut must be completely removed from the eye at the end of surgery.

Only medical personnel who have received training in the safe handling of the device are permitted to use the device.

Contraindications

F4H5® WashOut is not permitted to be used in severely injured eyes because it may escape into the orbital cavity. F4H5® WashOut is not permitted to be used in eyes with uveitis because of their increased tendency to become inflamed.

Side effects and complications

The side effects and complications associated with F4H5® WashOut include corneal complications, secondary glaucoma, elevated intraocular pressure, hypotony, intraocular inflammations, ERM formation, emulsification, cataract, and secondary proliferation.

Commercial form

F4H5® WashOut is available in 5-mL units in vials.

Storage information

F4H5® WashOut is stored at room temperature.

Sterilisation

Sterility is ensured through sterile filtration and autoclaving. The device is autoclaved in the primary container (vial).

Warnings and precautions

- F4H5® WashOut should be injected with the enclosed, sterile, disposable syringe. When using other sterile disposable syringes, ensure that F4H5® WashOut is injected immediately after filling. Otherwise the syringe can become stiff to move, thus preventing even application. The reason for this is the coating of conventional disposable syringes with silicone, which is dissolved by F4H5® WashOut.
- Do not use this device after the expiry date. The expiry date is shown on the fold-ing box.
- Vials are only intended for single use on one patient. Repeated use can lead to microbial contamination, which may cause a severe infection in the patient.

Please contact Fluoron GmbH if unexpected reactions or complications should occur during or following application that are suspected of being associated with F4H5® WashOut.

Interactions

F4H5® WashOut and silicone oils will mix at any ratio. This enables large volumes of silicone oil to be effectively removed. However, due to its solubility, F4H5® WashOut should not be directly replaced with silicone oil because this will not ensure a clearly defined boundary. There are currently no known interactions with other substances.

Other information

Any residual F4H5® WashOut in the vitreous chamber may be completely removed with the use of a backflush handpiece.

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FLUORON GmbH