F-Octane

F-Decalin

BG	1	IT	13
CZ	2	LT	14
DE	3	LV	15
DK	4	NL	16
EE	5	NO	17
EN	6	PL	18
ES	7	PT	19
FI	8	RO	20
FR	9	RU	21
GR	10	SE	22
HU	11	SI	23
IS	12	SK	24

F-Octane / F-Decalin

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F-Octane / F-Decalin

Ultrapure perfluorocarbons

F-Octane/F-Decalin are colourless, sterile, high-density fluorocarbon compounds consisting exclusively of C-C and C-F bonds with no biologically active components. Based on the extraordinary stability of the C-F bonds, F-Octane/F-Decalin are chemically and physiologically inert, non-toxic, and laser-compatible. F-Octane/F-Decalin are low-viscosity and insoluble in water.

F-Octane/F-Decalin are very well suited for laser coagulation and cryotherapy based on their high physical and chemical stability. F-Octane/F-Decalin produced by Fluoron GmbH are characterised by their consistently high quality. F-Octane/F-Decalin are used as an aid in the manipulation of the retina during surgical treatment of retinal detachments.

Indications

F-Octane/F-Decalin are used as medical aids, especially in cases of:

- Unfolding detached retinas
- Giant tears
- Traumas
- Laser coagulation
- Cryotherapy
- Floating dislocated lenses
- Temporary tamponades



Composition

F-Octane: 100 % perfluorooctane (C_8F_{18})
 F-Decalin: 100 % perfluorodecalin ($C_{10}F_{18}$)

Physical properties

Device	Density [kg/m ³ , 35 °C]	Refractive index [35 °C]
F-Octane	1,73	1,26
F-Decalin	1,91	1,31

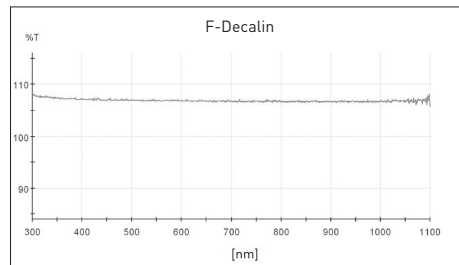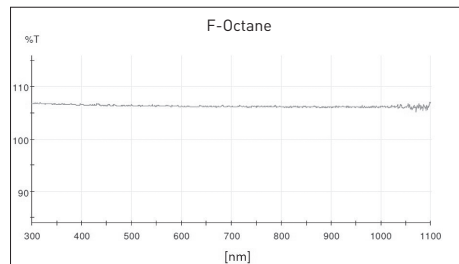Spectral transmission

Operating instructions

F-Octane/F-Decalin are intended only for ocular application; they are applied at a dose determined by an ophthalmic surgeon for a defined indwelling time.

Preparation

F-Octane/F-Decalin in vials are drawn up undiluted into a sterile syringe with a suitable cannula attached.

F-Octane/F-Decalin in sterile glass syringes may be used immediately.

F-Octane/F-Decalin are injected precisely above the optic disk following partial or complete vitrectomy. Due to the low viscosity, standard cannulas are able to be used. In the case of giant tears, the fluid must not be permitted to penetrate the subretinal tissue. At the end of surgery, F-Octane/F-Decalin must be completely removed from the eye and replaced with a long-term tamponading agent, as necessary.

Only medical personnel who have received training in the safe handling of the device are permitted to use the device.

Contraindications

F-Octane/F-Decalin are not permitted to be used as long-term tamponades due to their physical characteristics.

Side effects and complications

The side effects and complications associated with F-Octane/F-Decalin include incomplete removal, formation of „sticky oil“ in contact with silicone oil, subretinal penetration, corneal complications, elevated intraocular pressure, hypotony, intra-ocular inflammations, emulsification, and cataract.

Commercial form

F-Octane/F-Decalin are available in 5- and 7-mL units in vials and glass syringes.

Storage information

F-Octane/F-Decalin are stored at room temperature.

Sterilisation

Sterility is ensured through sterile filtration and autoclaving. The device is autoclaved in the primary container (vial or glass syringe).

Warnings and precautions

- Excessive instillation of F-Octane/F-Decalin in the posterior segment of the eye must be avoided.
- Prior to unfolding the retina using F-Octane/F-Decalin, all traction must be removed in order to prevent the additional weight of the perfluorocarbons from inducing iatrogenic retinal detachments or retinal breaks.
- Glass syringes are potentially fragile due to their material structure. It is recommended to use appropriate application systems.
- Do not use this device after the expiry date. The expiry date is shown on the folding box.
- Vials and syringes are only intended for single use on one patient. Repeated use can lead to microbial contamination, which may cause a severe infection in the patient.

Please contact Fluoron GmbH if unexpected reactions or complications should occur during or following application that are suspected of being associated with F-Octane/F-Decalin.

Interactions

Interactions with silicone oils may occur if F-Octane/F-Decalin are not completely removed (e.g. sticky oil). There are currently no known interactions with other substances.

Explanation of symbols

	Article number		Do not use if packaging is damaged
	Batch code		Store dry
	Manufacturer		Sterilised with steam
	Conformity mark and ID number of the Notified Body		Do not reuse
	Follow instructions for use		Do not resterilise
	Number of units		Store between 10° C – 30° C
	Use before		Crimped rim ampoule
			Syringe

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FLUORON GmbH