EasyGas[®] SF₆

EasyGas[®] C₂F₆

EasyGas[®] C₃F₈

BG	1	IT	15
CZ	2	LT	16
DE	3	LV	17
DK	4	MT	18
EE	5	NL	19
EN	6	NO	20
ES	7	PL	21
FI	8	PT	22
FR	9	RO	23
GR	10	RU	24
HR	11	SE	25
HU	12	SI	26
IE	13	SK	27
IS	14		

EasyGas® SF₆ / EasyGas® C₂F₆ / EasyGas® C₃F₈

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EasyGas® SF₆ / EasyGas® C₂F₆ / EasyGas® C₃F₈

EasyGas® SF₆/C₂F₆/C₃F₈ are colourless, odourless, and biocompatible gas mixtures. EasyGas® SF₆/C₂F₆/C₃F₈ are chemically and physiologically inert. The ready-to-use system manufactured by Fluoron GmbH is especially distinguished by its simple, quick, and safe handling. EasyGas® SF₆/C₂F₆/C₃F₈ are used following vitrectomy.

Indications

EasyGas® SF₆/C₂F₆/C₃F₈ are used following surgical treatment of macular holes. In addition, EasyGas® SF₆/C₂F₆/C₃F₈ are used in corneal grafting and as a long-term tamponade following surgical treatment of severe retinal detachments, especially in cases of:

- Retinal detachments with giant tears
- Retinal detachments without proliferation
- Retinal detachments with proliferative vitreoretinopathy (PVR)
- Retinal detachments in cases of proliferative diabetic retinopathy (PDR)
- Traumatic retinal detachments

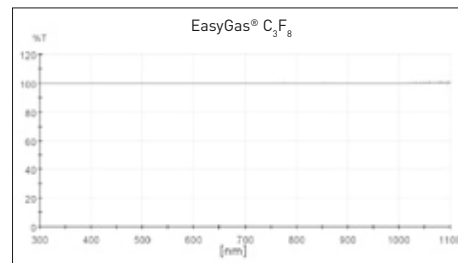
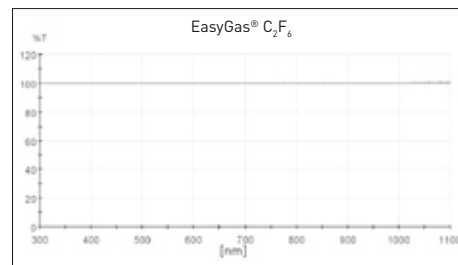
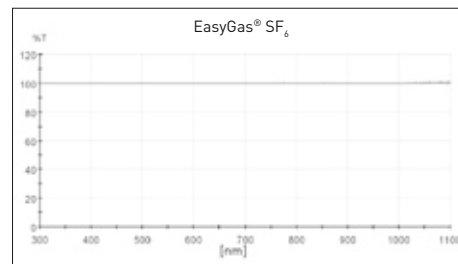
Composition

Device	Gas concentration	Synth. air
EasyGas® SF ₆	20 % sulphur hexafluoride	80 %
EasyGas® C ₂ F ₆	16 % hexafluoroethane	84 %
EasyGas® C ₃ F ₈	12 % octafluoropropane	88 %

Physical properties

Device	Density [kg/m³, 25° C]
EasyGas® SF ₆	2,34
EasyGas® C ₂ F ₆	1,95
EasyGas® C ₃ F ₈	2,01

Spectral transmission

Operating instructions

EasyGas® SF₆/C₂F₆/C₃F₈ are intended only for ocular application; they are applied at a dose determined by an ophthalmic surgeon for a defined retention time.

Preparation

EasyGas® SF₆/C₂F₆/C₃F₈ are heavier than air. In order to prevent the gas from inadvertently escaping, the syringe must be held or stored upright and not horizontally following removal of the stopper and syringe filter or with the opening pointed downward until it is connected to the adapter system.

EasyGas® SF₆/C₂F₆/C₃F₈ are used as a long-term tamponade medium following vitrectomy. The gas may expand slightly within the first 48 hours.

EasyGas® SF₆/C₂F₆/C₃F₈ are not removed, but instead are exhaled over a period of 1 to 8 weeks and replaced by bodily fluids.

Device	EasyGas® SF ₆	EasyGas® C ₂ F ₆	EasyGas® C ₃ F ₈
Effective tamponade period [days]	6	15	30
Retention time [weeks]	1–2	4–5	6–8

In corneal grafting applications, EasyGas® SF₆/C₂F₆/C₃F₈ may be used in the anterior chamber to press and hold the Descemet membrane in place.

Only medical personnel who have received training in the safe handling of the device are permitted to use the device.

Contraindications

Although EasyGas® SF₆/C₂F₆/C₃F₈ are inert gases, they are not recommended for use in cases of hypersensitivity to any of the components.

Side effects and complications

Side effects and complications associated with a gas tamponade include corneal complications, elevated intraocular pressure, hypotony, intraocular inflammation, cataract, subretinal gas, retinal vascular occlusions, peripheral scotoma, and retinal tears following macular hole treatment.

Commercial form

EasyGas® SF₆/C₂F₆/C₃F₈ are supplied as a ready-to-use system with a filled volume of 40 mL. Variations in pressure and temperature may slightly alter the gas volume in the syringe. This has no negative impact on the quality and functionality of the device.

Storage information

Store EasyGas® SF₆/C₂F₆/C₃F₈ at room temperature (10° C – 30° C). Severe variations in pressure and temperature must be avoided.

Sterilisation

Sterility is ensured through gamma sterilisation. The device is sterilised in the primary container (syringe) with the secondary packaging.

Warnings and precautions

- Check to ensure that the vitreous body has been completely removed prior to application of EasyGas® SF₆/C₂F₆/C₃F₈.
- Patients with a gas tamponade are not permitted to be subjected to any variations in pressure (flying, diving, etc.).
- If administering nitrous oxide as an anaesthetic, discontinue administration at least 20 minutes prior to application of EasyGas® SF₆/C₂F₆/C₃F₈.
- Remind the patient to wear the provided wristband indicating the corresponding retention time for the EasyGas® SF₆/C₂F₆/C₃F₈ device in use.
- Do not use the device if the colour of the sterilisation indicator adhered to the sterilisation pouch has not changed from orange to red.
- Do not use this device after the expiry date. The expiry date is shown on the folding box.
- Syringes are only intended for single use on one patient. Repeated use can lead to microbial contamination, which may cause a severe infection in the patient.
- Special note for the product EasyGas® SF₆: Nd:YAG laser treatment must not be performed on the gas-filled patient eye.
- Not for use by nursing mothers, pregnant women or children.

Please contact Fluoron GmbH if unexpected reactions or complications should occur during or following application that are suspected of being associated with EasyGas® SF₆/C₂F₆/C₃F₈.

Interactions

There are currently no known interactions with other substances.

Disposal

EasyGas® must be disposed of in accordance with the regulations of your organization. WHO recommendations: "Safe management of wastes from health-care activities, 2014, WHO".

Explanation of symbols

Manufacturer

Use-by date

Batch code

Sterilised with gamma sterilisation

Do not resterilise

Do not use if packaging is damaged

Do not reuse

Store between 10° C – 30° C

Follow instructions for use

Date of the surgery

Wristband valid until

Patient card valid until

No diving and snorkeling

No mountaineering

No travelling by plane

No treatment with nitrous oxide

Syringe

Conformity mark and ID number of the Notified Body

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FLUORON GmbH